

CONTRAINDICATIONS POWER NECK MASSAGER

1. Before using, please consult a doctor if:

- you suffer from heart diseases or high blood pressure
- you suffer from infectious diseases
- you suffer from a cancer
- you have a diagnosis of schizophrenia
- you have an open wound, cut, redness, infections, burn or bruise on the skin
- you are pregnant, in the period of menstruation and incessant fever
- you have a sensitive skin, angiectasia or were hospitalized
- you had a neck surgery
- your joints and muscles can be described as slow to warm up

2. Do not use the Massager when driving a car.

3. Do not disassemble the device yourself. If necessary, contact the service center.

4. Do not use, charge or store the device in hot or humid environment or in direct sunlight in order to avoid failure and malfunction.

5. Do not allow children and disabled people to use the device unattended.

6. Do not leave the massager under water or in rooms that provoke corrosion.

7. In case of malfunction stop using the device immediately.

8. Do not drop or hit the device.

9. Do not put heavy objects on the massager.

10. The massager is not a replacement for treatment or doctor's recommendations.

11. The massager shall not be used for diagnosis, treatment or for commercial purposes.

12. Do not use the massager during charging.